∴ HEINE NT300 Charger

HEINE Optotechnik GmbH & Co. KG
Dornierstr. 6 · 82205 Gilching · Germany
E-Mail: info@heine.com · www.heine.com

MED 0613 2020-06-26

HEINE NT300 Charger

DEUTSCH	3
ENGLISH	7
FRANÇAIS	11
ESPAÑOL	15
ITALIANO	19
SVENSKA	23
NEDERLANDS	27
DANSK	31
SUOMI	35
PORTUGUÊS	39

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Please read and follow these instructions for use and keep them for future reference.

General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle.

The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criteria: colour temperature, useful service life, safety, optical quality and performance. The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Intended Use

The HEINE NT300 charger is designed exclusively for charging HEINE BETA rechargeable handles, HEINE BETA Slim rechargeable handles and HEINE Laryngoscope rechargeable handles that must contain a HEINE rechargeable battery. It should only be operated by qualified medical personnel within a professional healthcare facility.

Warnings and Safety Information

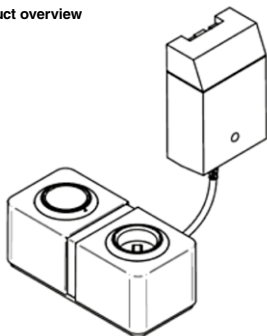
CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! This symbol indicates valuable advice in terms of set up, operation or maintenance, as applicable. Notes are important, but not related to hazardous situations.

Product overview

Primary plug:

EU

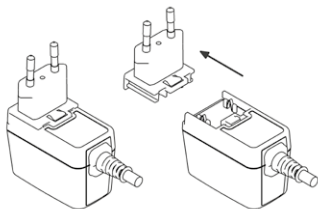
USA/Japan

UK

Australia

The primary plug is interchangeable. This allows a world wide use. The procedure of changing the plugs is as per the pictures.

⚠ To prevent mechanical and electrical breakdown, never install or store the charger in the following areas:

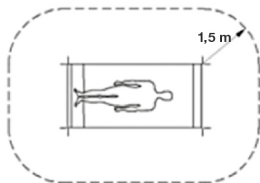
- areas exposed to high moisture or condensation
- areas exposed to extreme environmental conditions
- areas subject to constant vibrations
- areas subject to severe temperature fluctuations
- outdoors

Installation

⚠ Do not place the NT300 on soft furniture, cloth or carpet.

Do not insert any objects in any opening in the housing.

The HEINE Charger is intended for installation in medical areas outside the patient environment (at least 1.5 metres from the patient or patient support pursuant to IEC 60601-1, see figure).

Installation, operation and use

The HEINE NT300 is compatible with the following rechargeable handles:

- BETA L rechargeable handle
- BETA NT rechargeable handle
- BETA Slim rechargeable handle
- Standard F.O. laryngoscope handle
- Short F.O. laryngoscope handle
- F.O. angled handle
- Small F.O. laryngoscope handle

 The NT300 may only be used at the mains voltage specified on the identification label. Unplug the unit from the mains if not in use over long periods.

In the event of fluid ingress in the housing, the charger should be taken out of service and checked by an authorised person.

The NT300 must not be used in rooms where an explosion hazard is present.

Follow the instructions for the charger and rechargeable handles.

 The following handles are **not** compatible with the NT300 charger:
BETA TR, BETA R, BETA cord handle, mini3000 battery handle.

Plug the mains transformer into the mains and the NT300 is ready for use.
NT300 ensures that batteries are kept in good condition irrespective of handle size.

 When replacing the handles in the charging compartments, ensure that the internal contacts in the compartment are clean and that the instruments are switched off.

Instruments placed next to each other must not touch each other.

If different-sized handles are used, it is possible to fit adaptors so that each handle is held in place correctly.

Reduction rings for:

BETA Handle
F.O. angled handle
Standard F.O.
Short F.O. laryngoscope handle

Small F.O. laryngoscope handle
(optional)

BETA Slim rechargeable handle
(optional)

Ideally, the rechargeable handle should always be replaced and stored in the charger. Handles cannot be overcharged.

 The rechargeable battery can only be charged when fitted to the handle.

 The NT 300 must be disconnected from the mains supply when the adaptors are being fitted.

If the second charging compartment is not in use it can be sealed off by the dust cap which is supplied with the unit.

The pulsating light around the charging compartment shows that current is flowing and that the battery is being recharged.

When the light is permanently lit, the charging sequence is completed. The battery in the handle is fully-charged and is ready to supply maximum power.

 If the light is switched off, this indicates that the charging compartment is empty or the unit is defective.

Hygienic reprocessing

The instruction is available:

- at www.heine.com
- in a paper version which you can request from the address listed

Maintenance

The charger is maintenance free.

 Do not open the power adapter (risk of electric shock).

Service

The device has no components serviceable by the end-user.

Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

Used or defect batteries must be disposed adequately. Avoid skin contact with defect batteries.

General Notes and Warnings

The HEINE NT300 charger is a high-quality piece of equipment. Always handle with care.

 Check the correct operation of the product at regular intervals. Do not use the equipment if it shows visible signs of damage.

Do not use the device in hyperbaric chambers, in explosive or oxygen loaded environments!

Do not use the device near strong magnetic fields like MRI scanners.

We can only guarantee the performance of this device when fitted and used with original HEINE accessories and spare parts. Otherwise the HEINE Warranty is terminated.

The NT300 must not be charged using rechargeable batteries from other manufacturers.

Do not manipulate or modify the product!

The LED indicates that the power adapter is ready for use.

Do not pull the cable to disconnect the power adapter from the mains.

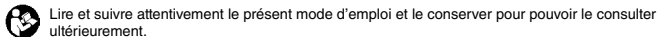
Do not store or use the charger outdoors.

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the NT300 including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	See attachment
Test	Compliance
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Passed
Voltage Fluctuations/Flicker	Passed

Technical specification

Mains supply	100 – 240 V AC / 50 – 60 Hz
Current consumption	max. 30 VA
Output	15 V DC, 2 A
Fuse	integral overload protection
Charging current	max. 1.5 A for BETA L max. 0.5 A for other HEINE rechargeable batteries
Charging time	BETA L rechargeable battery -> approx. 2 hrs other HEINE rechargeable batteries -> up to 4 hrs
Safety class	II
Protection class	IP 20
<i>Environmental conditions for operation</i> Temperature Humidity Air pressure	+10 °C to +40 °C 10 % to 75 % r.h. 700 hPa to 1060 hPa
<i>Environmental conditions for transport and storage</i> Temperature Humidity Air pressure	-20 °C to +50 °C 10 % to 95 % r.h. 500 hPa to 1060 hPa

Erläuterung der verwendeten Symbole
Explanation of utilized symbols
Explication des symboles utilisés
Explicación de los símbolos utilizados
Spiegazione dei simboli utilizzati
Förklaring av symboler som används
Verklaring van de gebruikte symbolen
Forklaring af de anvendte symboler
Käytettyjen symbolien selitys
Explicação dos símbolos utilizados

	CE-Kennzeichnung kennzeichnet die Übereinstimmung mit der Europäischen Medizinprodukterichtlinie 93/42 EWG. The CE mark indicates that the product complies with the European medical device directive 93/42/EEC. Le marquage CE indique la conformité à la directive européenne 93/42/CEE relative aux dispositifs médicaux. El marcado CE indica la conformidad con la directiva europea 93/42 /CEE relativa a los productos sanitarios. Il marchio CE indica la conformità con la direttiva europea sui dispositivi medici 93/42 CEE. CE-märkning markerar en överensstämmelse med det europeiska direktivet för medicinska produkter 93/42 EEG. CE-markering duidt de overeenstemming aan met de Europese Richtlijn betreffende medische hulpmiddelen 93/42 EEG. CE-mærkningen angiver overensstemmelse med det europæiske direktiv 93/42/EØF om medicinsk udstyr. CE-merkintä tarkoittaa, että laite vastaa eurooppalaisen lääkinnällisiä laitteita koskevan standardin 93/42 ETY vaatimuksia. O símbolo CE identifica a concordância com a Diretriz Europeia para Dispositivos Médicos 93/42/CEE.
	Katalog- oder Bestellnummer Catalogue- or order number Numéro de catalogue ou de commande Número de catálogo o de pedido Codice catalogo e di dell'ordine numero Katalog- eller Beställningsnummer Catalogus- of Bestelnummer Katalog- eller Ordrenummer Luettelo- tai viitenumero Número de catálogo ou do pedido
	Serienummer Serial number Numéro de série Número de série Numero di serie Seriennummer Seriennummer Seriennummer Sarjanumero Número de série

	Herstellere Manufacturer Fabricant Fabricante Produttore Tillverkare Fabrikant Producent Valmistaja Fabricante
	Hersteldatum Date of manufacture Date de fabrication Fecha de fabricación Data di produzione Tillverkningsdatum Productiedatum Produktionsdato Valmistuspäivä Data de fabricação
	Getrennte Sammlung von Elektro- und Elektronikgeräten (Europäische WEEE Richtlinie) Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions. (European Waste Electrical and Electronic Equipment Directive, WEEE) Tri sélectif des appareils électriques et électroniques (Directive européenne DEEE) Desechado separado de aparatos eléctricos y electrónicos Directiva Europea RAEE) Raccolta differenziata di apparecchi elettrici ed elettronici (direttiva europea RAEE) Separat insamling av elektriska och elektroniska apparater (det europeiska WEEE-direktivet) Gescheiden inzameling van elektrische en elektronische apparaten (Europese AEEA richtlijn). Separat indsamling af elektrisk og elektronisk udstyr (det europæiske WEEE-direktiv) Sähkö- ja elektroniikkalaitteille tarkoitettu erillinen keräyspiste (eurooppalainen WEEE-standardi) Coleção separada de aparelhos elétricos e eletrônicos (Diretrizes Europeias WEEE)
	Zulässiger Temperaturbereich in °C für Lagerung und Transport Temperature limits in °C for storage and transport Plage de température admise en °C pour le stockage et le transport Rango de temperatura permitida en °C para almacenar y transportar el producto Temperatura ammessa in °C per conservazione e trasporto Tillåtet temperaturintervall i °C för lagring och transport Toegestane temperaturen in °C voor opslag en transport Tilladt temperaturområde i °C ved opbevaring og transport Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°C) Gama de temperatura permitida em °C para o armazenamento e transport

	Zulässiger Temperaturbereich in °F für Lagerung und Transport Temperature limits in °F for storage and transport Plage de température admise en °F pour le stockage et le transport Rango de temperatura permitida en °F para almacenar y transportar el producto Temperatura ammessa in °F per conservazione e trasporto Tillåtet temperaturintervall i °F för lagring och transport Toegestane temperatueren in °F voor opslag en transport Tilladt temperaturområde i °F ved opbevaring og transport Näyttää pakkauksen sallitun säilytys- ja kuljetuslämpötilan (°F) Gama de temperatura permitida em °F para o armazenamento e transporte
	Zulässige Luftfeuchtigkeit für Lagerung und Transport Humidity limitation for storage and transport Humidité admise pour le stockage et le transport Humedad del aire permitida para almacenar y transportar el producto Umidità atmosferica ammessa durante il trasporto e la conservazione Tillåten luftfuktighet för transport och lagring Toegestane luchtvochtigheid voor opslag en transport Tilladt luftfugtighed ved opbevaring og transport Sallittu ilmankosteus kuljetuksen ja varastoinnin aikana Umidade do ar admissível para o armazenamento e transporte
	Zulässiger Luftdruck für Lagerung und Transport Pressure limitation for storage and transport Pression atmosphérique admise pendant le transport et le stockage Presión de aire permitida para almacenar y transportar el producto Pressione atmosferica ammessa durante il trasporto e la conservazione Tillåten lufttryck för lagring och transport Toegestane luchtdruk voor opslag en transport Tilladt lufttryk ved opbevaring og transport Sallittu ilmanpaine kuljetuksen ja varastoinnin aikana Pressão do ar admissível para o armazenamento e transporte
	Bedienungsanleitung lesen Instructions for use Mode d'emploi Manual de instrucciones Istruzioni per l'uso Bruksanvisning Gebruiksaanwijzing Brugsanvisning Käyttöohjeet Instruções de utilização
	Wechselstrom (AC) Alternating current (AC) Courant alternatif (CA) Corriente alterna (CA) Corrente alternata (AC) Växelström (AC) Wisselstroom (AC) Vekselstrøm (AC) Vekselstrøm (AC) Vaihtovirta (AC) Corrente alternada (CA)

	Sicherung Fuse Fusible Fusible Fusibile Säkring Zekering Sikring Sulake Fusivel
	Vorsicht Bruchgefahr! Fragile, handle with care! Fragile ! Manipuler avec soin Atención. Frágil. Attenzione: pericolo di rottura! Försiktigt! Risk för brott Voorzichtig, kans op breuk! Forsigtig, risiko for brud! Ømtålig, behandles forsigtig! Varo sårkymisvaaraa! Frágil, manuseie com cuidado!
	Trocken lagern! Keep dry! Conserver au sec ! Conservar en un lugar seco! Evitare ambienti umidi! Förvaras torrt! Droog bewaren! Opbevares tørt! Säilytetään kuivassa paikassa! Armazenar em ambiente seco!
	Nur in geschlossenen Räumen benutzen For indoor use only Utiliser uniquement dans des locaux fermés. Sólo utilizar en espacios cerrados Utilizzare solo in ambienti chiusi Får endast användas i slutna rum Uitsluitend in afgesloten ruimten toepassen Må udelukkende benyttes i lukkede rum Käyttö on sallittu ainoastaan suljetuissa tiloissa. Utilizar apenas em espaços fechados
	Gebrauchsanweisung verbindlich befolgen. (Hintergrundfarbe: blau, Vordergrundfarbe: weiß) Follow instructions for use! (Background color: blue, foreground color: white.) Suivre le mode d'emploi. (Couleur de fond : bleu ; couleur du premier plan : blanc) Seguir obligatoriamente las instrucciones de uso. (Color de fondo: azul, color de primer plano: blanco) Attenersi obbligatoriamente alle istruzioni per l'uso. (Colore dello sfondo: blu, colore in primo piano: bianco) Bruksanvisningen ska alltid följas. (Bakgrundsfärg: blå, förgrundsfärg: vit) De gebruiksaanwijzing is bindend en dient gevolgd te worden. (achtergrondkleur: blauw, voorgrondkleur: wit) Følg altid brugsanvisningen. (Baggrundsfarve: Blå; forgrundsfarve: Hvid) Käyttöohjetta on noudatettava tarkasti. (Taustaväri: sininen, etualan väri: valkoinen) Leia e cumpra as instruções de uso e guarde-as para consulta futura. (fundo: azul; primeiro plano: branco)

	Genehmigungszeichen von SIQ Approval mark of SIQ Marque d'homologation SIQ Sello de autorización de SIQ Marchio di conformità SIQ Godkännandesymbol från SIQ Toestemmingsteken van SIQ Godkendelsesmærke fra SIQ SIQ-laitoksen hyväksymismerkintä Símbolo de aprovação da SIQ
	Markenzeichen des Herstellers Trade mark of the manufacturer Sigle du fabricant Marca del fabricante Marchio del produttore Tillverkarens märkessymbol Merkteken van de fabricant Producentens varemærke Valmistajan tavaramerkki Símbolo da marca do fabricante
ta40/F	Bemessungs-Umgebungstemperatur / Wandler Temperaturklasse Rated ambient temperature / converter temperature class Température ambiante nominale / classe de température du transformateur Temperatura ambiente de medición / transformador de clase de temperatura Temperatura ambiente di taratura / trasformatore classe Nominell omgivningstemperatur / omvandlare temperaturklass Vaststelling omgevingstemperatuur / omzetter temperatuurklasse Dimensionerings-omgevingsstemperatuur / transformer temperatuurklasse Mittausympäristön lämpötila / muuntajan Lämpötilaluokka Temperatura de dimensionamento e ambiente / conversor classe de temperatura
	Gerät der Schutzklasse II Class II equipment Appareil de classe de protection II Aparato de clase de protección II Apparecchio di classe di protezione II Apparat i skyddsklass II Apparaat van beschermingsklasse II Apparater i sikkerhedsklasse II Suojausluokan II laite Aparelho da classe de proteção II
IPx0	IP-Code (Schutzart) IP code (Ingress Protection Rating) Indice IP Índice IP Cifre di identificazione IP IP-sifra IP-kencijfers IP-kode (beskyttelsestype) IP-numerot Números de identificação IP
	Genehmigungszeichen für Australien Approval mark of Australia Marque d'homologation pour l'Australie Licencia para Australia. Marchio di conformità per Australia Godkännandetecken för Australien Toestemmingsteken voor Australië Godkendelsesmærke for Australien Australian hyväksymismerkintä Símbolo de aprovação para Austrália
	Unique Device Identification