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Important Information for users of EICKEMEYER-Fiber-Light-Guides

USA

Thank you for your expression of confidence in the EICKEMEYER Brand name. Follow all safety instructions. EICKEMEYER accepts no liability for any damages resulting from faulty use or handling of the light guides. Read this instruction manual before use.

General Information: Light Guides carry the light energy from the light sources to the endoscopes

Caution: The tip of the Light Guide is hot. Never put the tip on flammable materials, on patient wrappings, or directly on the human skin.

Caution: Light guides are supplied non-sterile. It is user's responsibility to clean, disinfect or sterilize the light guides.

Caution: Never damage the protective outer hose of the Light Guide. Protect the Light Guide from sharp bending which may damage the inner fiber bundle.

Caution: Select the Light Guide fiber diameter correctly. Avoid mismatch. There is a heat danger due to unintended reflection by a mismatch at the telescope site. Thin telescopes for arthroscopy, cystoscopy, resectoscopy, sinuscopy, require 3.5 mm Light Guides. Thick telescopes for laparoscopy require 4.9mm Light Guides.

Caution: Follow carefully all instructions of the disinfection and detergent solutions manuals. Select only the recommended solution ratio, and application timing. The Light Guides may be damaged if these instructions are disregarded, if too strong mix ratio or too much soak time are applied.

Caution: Warranty is void if these instructions are not followed. Never force the adapters, never bend the light guide.

Caution: Never use the standard glued Light Guides with high temperature Xenon or Metal Halide light sources. Select the EICKEMEYER Series HTT-BlueLine® high temperature Light Guides for Xenon and Metal Halide or other high-temperature light sources. Warranty is void for heat damages on the Light Guide from high temperature light sources.

Information: The EICKEMEYER Light Guides are autoclavable up to 134°C or 274°F, as well as sterilizable by formaldehyde, ethylene oxide or plasma. Hot air sterilization is not allowed. Chemical and thermal methods are possible.

Information: Before use test the Light Guides. Hold one end to the surface of a lamp. Check the other end. Dark spots are the signs of broken light fibers. Should the dark spot cover more than 20-25% of the surface, discard the guide. The screwed end-adapters are reusable. Save them.

Disposal: No special care required

CE Marking: Light Guides comply with the Medical Device Directive (MDD) 93/42/EEC. All light numbers are marked with a serial

Warranty: 2 years from the date of purchase.

Our lab, or our authorized distributors will check the warranty claims and we will provide a replacement free of charge in case of approved claims. All freight charges are at user's account.

Caution: Do not attempt to make repairs on the Light Guides. Warranty is void if user tries to alter or tamper with the Light Guide.

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