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Gekauft bei

am

HEINE HSL 150, alpha+ HSL 150, HSL 10x, Hand-held Slit Lamp with Accessories

CE 93/42/EWG/CEE/EEC

Instructions

HEINE HSL 150/alpha+ HSL 150 Hand-held Slit Lamp with Accessories

Before using the HEINE HSL150/alpha+ HSL 150 read these instructions carefully and keep them for future reference.

Safety Information

Range of applications: This instrument should only be used for examination of the anterior segment of the eye.

Safety in use: Use only suitable power sources which are approved for medical use. The supply voltage must correspond to the bulb voltage.

We can only guarantee the performance of the HSL 150/ alpha+ HSL 150 if original HEINE accessories are used. Do not use the HSL 10x as an loupe. Do not look into the sun with the HSL 10x.

Function

HSL 150/alpha+ HSL 150:

On the HSL 150/alpha+ HSL 150 the slide control (6) can be used to adjust the size of the slit. The loupe gives magnification in the left and right working position.

HSL 10x:

With the HSL 10x, the magnification is increased to 10x. The instrument is used in the same way as without the accessory.

Using the HSL 150/alpha+ HSL 150 (7):

- Connect the HSL to a handle.
- Swivel the loupe (1) to the left stop to examine with the right eye, and vice versa.
Note: Examine the patient's right eye - as you would with a direct ophthalmoscope - with your right eye and vice versa.
- Approach the patient from a 30° angle to a distance of about 4cm.
- Looking through the loupe, sweep the eye with the slit by turning the handle.
- The image can be fine-focused by adjusting the distance from the HSL to the patient's eye.
- The blue filter is selected by rotating the filter attachment (2) by 180°. Pull the attachment upwards to remove.

HSL 10x (8):

see figures ① ② ③

Changing the bulb

Important Note: We can only guarantee the safety and performance of this instrument if original HEINE bulbs are used.

- Remove the HSL from the handle.
- Unscrew the connector (3) and pull out the bulb (4).
- Wipe the glass of the bulb clean with a soft, clean cloth.
- Insert the bulb so that the pin (5) coincides with the groove.
- Screw the connector back in.

Cleaning and sterilizing

Use only a damp cloth with soap or alcohol to wipe the housing clean. Use an optical cleaning cloth to clean glass surfaces. Sterilize only by ethylene oxide. Do not immerse in fluids.

Accessories and spare parts

HEINE XHL® halogen bulb 2,5V	X-01.88.098
HEINE XHL® halogen bulb 3,5V	X-02.88.099
Replacement Filter attachment HSL 150	C-00.14.605
Replacement Filter attachment alpha+ HSL 150	D-00.81.502
HEINE HSL 10x Loupe attachment	C-00.14.606

Warranty

We guarantee this device for two years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of normal wear and tear, incorrect use, the use of non-original HEINE spare parts, or non-authorized modification or repairs. Further claims are excluded. For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

Bought from

date