



Manual

Cautery instrument

324050

1 Starting the device

Please put on the desired tip and connect the unit with the power supply cable on the mains.

By pressing the ON/OFF button the unit will heat and you will be able to work.

As soon as you have stopped pushing the button, the unit will not heat any longer.

The tips are made of special heat resistant steel and are easy to clean and disinfect.

2 Technical details

Power: 100W

Voltage: 230V/AC

Heat-Up Time: 6 - 10 seconds

Weight: 600 g

3 Content

Cautery instrument	1
1 Starting the device	2
2 Technical details	2
3 Content	3
4 Contact	4

GERMANY

EICKEMEYER KG
Eltastraße 8
78532 Tuttlingen
T +49 7461 96 580 0
F +49 7461 96 580 90
E info@eickemeyer.de
www.eickemeyer.de

ITALY

EICKEMEYER S.R.L.
Via G. Verdi, 8
65015 Montesilvano (PE)
T +39 085 935 4078
F +39 085 935 9471
E info@eickemeyer.it
www.eickemeyer.it

CANADA

EICKEMEYER Inc.
250 Briarhill Dr.
Stratford, Ont. Canada
N5A 7S2
T +1 519 273 5558
F +1 519 271 7114
E info@eickemeyervet.ca
www.eickemeyervet.ca

NETHERLANDS

EICKEMEYER B.V.
Bedrijventerrein
Pavijen-West Bellweg 44
4104 BJ Culemborg
T +31 345 58 9400
F +31 345 58 9401
E info@eickemeyer.nl
www.eickemeyer.nl

SWITZERLAND

EICKEMEYER AG
Sandgrube 29
9050 Appenzell
T +41 71 788 23 13
F +41 71 788 23 14
E info@eickemeyer.ch
www.eickemeyer.ch

DENMARK

EICKEMEYER ApS
Lysbjergvej 6, Hammelev
6500 Vojens
T +45 7020 5019
F +45 7353 5019
E info@eickemeyer.dk
www.eickemeyer.dk

POLAND

EICKEMEYER Sp. z o.o.
Al. Jana Pawła II 27
00-867 Warszawa
T +48 22 185 55 76
F +48 22 185 59 40
E info@eickemeyer.pl
www.eickemeyer.pl

UNITED KINGDOM

EICKEMEYER Ltd.
3 Windmill Business Village
Brooklands Close
Sunbury-on-Thames
Surrey, TW16 7DY
T +44 20 8891 2007
F +44 20 8891 2686
E info@eickemeyer.co.uk
www.eickemeyer.co.uk